



RMC MEDICAL

PDS-2000 **DECONTAMINATION TABLETOP**

Installation Instructions

1. Remove the mattress pad from the stretcher where the Decontamination Tabletop will be placed.
2. Lift the Decontamination Tabletop off the wall bracket.
3. Place Decontamination Tabletop onto stretcher ensuring it is centered from side to side and top to bottom.

Note: The drain end must have proper clearance so that the drain hose can be pulled down and inserted into the 15-gallon wastewater container.

4. Release black belts.
5. Allow straps to hang down on either side of the stretcher. Reconnect belt underneath the stretcher. Pull tight.

Note: If slack remains in belt even after pulling as tight as possible, it will be necessary to shorten one or both sides of the strap. This can be accomplished by releasing the straps and feeding the straps through the “S” type tightening clamp.

6. Pull the drainage hose down and insert into the 15-gallon waste water container.